



Installation manual

CabinAid Essential



Parts list

Your Cabinaid Essential box contains:

- Installation Manual
- Cabinaid Cup
- Cabinaid Ventilation Cover
- Cabinaid Cover
- AED mounting Bracket
- Cabinaid Padlock Mounting piece (If ordered)
- Cabinaid Divider Plates (if ordered)
- Fastening materials



3 X Tapping screw



3X Wall plug



4 or 6 X Cap screw Hexagon

(Depending on what has been ordered, used for the ventilation cover, AED mounting bracket and Cabinaid lock bracket)



1 X P clip




1 X small cross head tapping screw for P-clip

Cabinaid installation manual



Step 1: Determine the location where you want to place the Cabinaid AED cabinet. Please make sure the desired location is suitable for placement of the Cabinaid. It is useful to consult fire and safety regulations before determining a suitable location. If a stone wall will be used as a fixing location, please check whether the wall anchors are utilized prior to fixation.

Step 2: Place the Cabinaid in the correctly orientated position on the wall. Using a marker to mark the location of the 3 drilling holes. Please see the image below for the locations of the screw holes and correct orientation. After marking the holes put the Cabinaid storage compartment aside and use a drill to make the holes in the wall. If applicable, place the screw plugs in the holes. Finally, place the back of the Cabinaid cup against the wall and align the cabinet with the previously drilled holes. Now you can mount the Cabinaid AED cabinet to the wall.

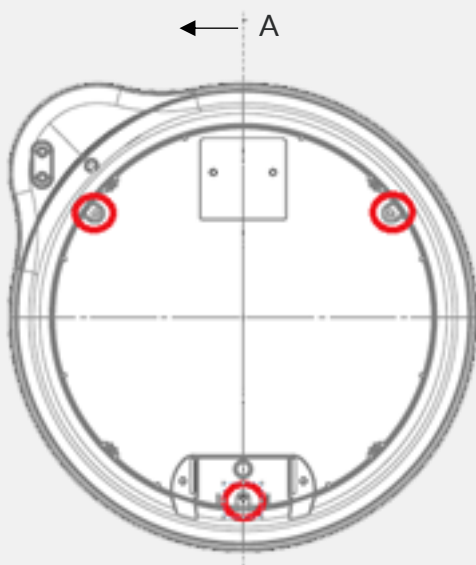

Step 3: Place the ventilation cover into the notch at the bottom of the Cabinaid Essential. Fasten the ventilation cover by using 2 hexagon cap screws. Use the delivered P-clip and the small cross head screw to attach the elastic band leading from the ventilation cover to the Cabinaid lid cover. The Cabinaid cover has an allocated point at the bottom where the screw needs to be placed. See the image on the next page for the location. Besides is a picture of how it should look when the elastic band is attached to the cover.

NOTE THAT THE ELASTIC BAND IS NOT TANGELED BEFORE ATTACHING

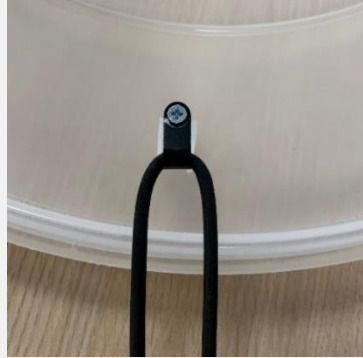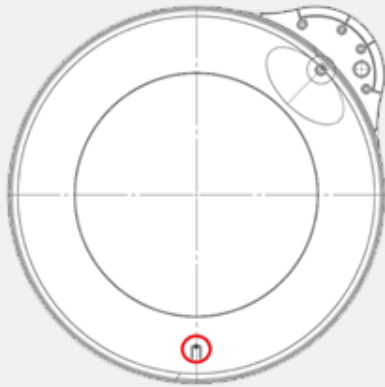

Step 4: Place the attached Cabinaid cover onto the Cabinaid cup. You should hear an audible click when the Cabinaid cover has sealed off the cup compartment. As a final check; check if the seal is closed all around the cup by applying pressure all-round the edge of the cover.


Your Cabinaid is now ready for storage!

Installing the AED hanging bracket

The AED Hanging bracket comes as standard with your Cabinaid AED Cabinet. The following instruction will show you optimally how to install the bracket.

Step 1: Open the Cabinaid AED cabinet by pulling the handle and remove the cover from the storage compartment.

Step 2: Take the AED hanging bracket and attach it to the indicated location using the 2 X hexagon cap screws that come with the hanging bracket. You can use the images below as a reference.

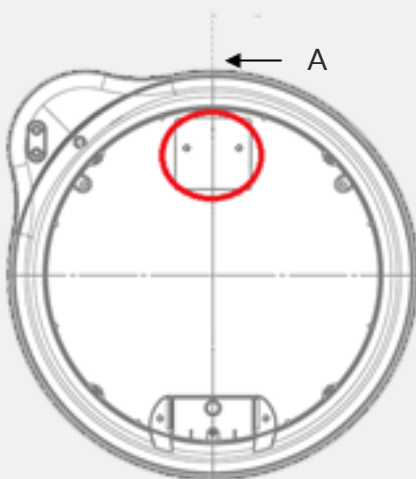

Step 3: By loosening and tightening the hexagon cap screws the bracket can moved up and down. Use this to adjust the hanging bracket to the desired height.


Installing the optional parts into the Cabinaid

Padlock mounting piece


If you decided to purchase a padlock mounting with your Cabinaid the following instruction will show you how to install the mounting piece.

Step 1: Open the Cabinaid by pulling the handle and remove the cover from the storage compartment.

Step 2: Take the mounting piece and attach its allocated location using the 2 X hexagon cap screws that come with the lock mounting piece. You can use the images below as a reference.

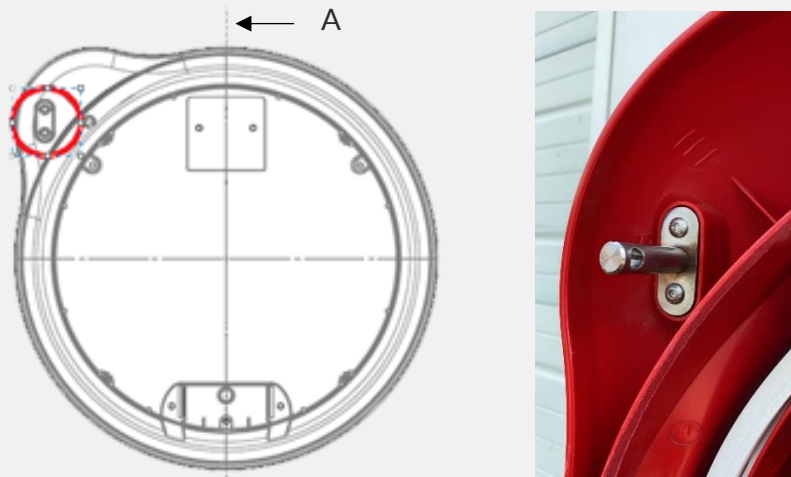

Step 3: Use a drill to drill a hole into the handle of the Cabinaid cover, drill from the back through the handle at the indicated location. See the image below for the location of the hole.

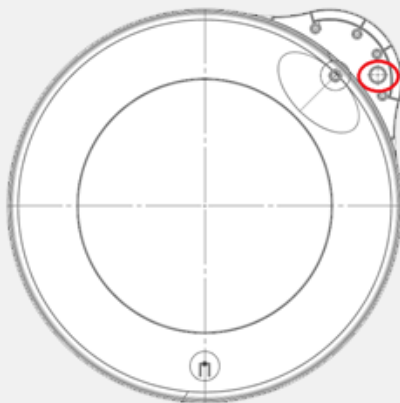

Step 4: Align the drilled hole in the handle with the mounting bracket and close the Cabinaid AED cabinet. You are now able to attach a locking mechanism of choice to the Cabinaid.

Divider plates

If you decided to purchase the divider plates with your Cabinaid the following instruction will show you how to install these plates.

Step 1: Open the Cabinaid by pulling the handle and remove the cover from the storage compartment.

Step 2: Take the divider plates and place them at the indicated locations using 2 X hexagon cap screws and 2 washers per plate. The screws and washers come with the divider plates. A maximum of 3 divider plates can be placed in an Cabinaid AED cabinet, you can use the image below as a reference.





Cabinaid

Celsiusstraat 15 • 6003 DG Weert • The Netherlands
T. +31 (0)495 760156 • info@cabinaid.com • www.cabinaid.com